



Consumer Information

TR Pro+™ is a non-sticky, transparent hydrogel formulated to assist skin following aesthetic procedures. TR Pro+™ can be applied to broken skin.

TR Pro+™ can be easily applied to all areas of the skin, including exposed areas like the face and neck as well as joints. TR Pro+™ can be applied to hairy areas without the need for shaving.

When used as directed, TR Pro+™ dries to form a thin biofilm protective layer that is gas permeable. The biofilm is beneficial because it hydrates and protects the skin from infection.

TR Pro+™ is used to soften skin, and to provide a cool soothing feeling.

There may be a brief mild stinging sensation before the cooling effect of the gel becomes apparent, at which time there may also be a feeling of skin tightening. The surface of the skin may also feel dry after applying TR Pro+™ so where appropriate, moisturiser should be applied between applications of TR Pro+™. The skin may feel slick when TR Pro+™ is washed off with water which is normal for hydrogel-containing products.

Intended Use

TR Pro+™ is used on skin following aesthetic procedures including ablative and non-ablative laser, skin needling, dermabrasion, LED, IPL and BBL light therapy, chemical peels, biopsies, and skin cancer removal.

TR Pro+™ is suitable for people with sensitive skin because it is neutral pH and does not contain any fragrances, or parabens.

Directions for use

Use following ablative and non-ablative laser procedures, skin needling, biopsy, and skin cancer removal

1. Day 1 gel application immediately post procedure (in clinic)

Rinse the treated areas with sterile water. Open a tube/sachet of TR Pro+™ and apply a generous layer across the affected area similar to the application of a mask (ie: approximately 2-mm thick, or the thickness of a 20-cent coin). The gel will take some time to absorb into the skin and can be covered with a loose dressing if necessary.

2. Days 2-5 gel application daily at home at night for next 4 days post procedure

Cleanse skin with a gentle cleanser during nightly showers for the first 5 days. Apply a generous layer of the gel across the affected area similar to what is described above. The gel will take some time to absorb into the skin and can be covered with a loose dressing if necessary. Moisturiser can be applied after showering in the morning and throughout the day as required to ensure the surface of the skin does not dry out.

3. Days 6-28 gel application twice per week at night

Subject to advice from your healthcare physician, TR Pro+™ can be applied for longer than 5 days. TR Pro+™ should be applied in the evening and left on overnight. The gel will take some time to absorb into the skin and can be covered with a loose dressing if necessary. Moisturiser can be applied after showering in the morning and throughout the day as required to ensure the surface of the skin does not dry out.

Sunscreen and other lotions can be applied during the day after showering.

Use for other conditions

Ensure that the skin is clean and dry. In the evening, apply TR Pro+™ to the affected area. The gel will take some time to absorb into the skin and can be covered with a loose dressing if necessary. Moisturiser can be applied after showering in the morning and throughout the day as required to ensure the surface of the skin does not dry out.

TR Pro+™ can be covered loosely with a dressing but should not be covered with anything that is fully occlusive.

Recommended duration of use

TR Pro+™ is recommended to be used for the duration of the time it takes the skin to return to a normal state. Clinical trial results suggest that improvements in skin quality continue for as long as the product is used. Where scarring may occur, TR Pro+™ can continue to be used for up to 90 days as the hydrating and partial occlusive properties of TR Pro+™ may have a positive benefit for normal scar formation.

Caution

For external use only. Keep out of reach of children

TR Pro+™ should not be applied to burns

TR Pro+™ should not be placed in contact with mucous membranes or the eyes

TR Pro+™ should not be applied over other skin treatments without the advice of your healthcare physician

TR Pro+™ should not be used with any other medicated lotions, creams, powders or solutions unless directed by your physician

A moisturiser can be applied as required between applications of TR Pro+™ to ensure the skin surface does not dry out

TR Pro+™ should not be used following a cosmetic procedure with any topical products containing alpha-hydroxy acids, salicylic acid, and vitamins C or E for the duration of application of the gel

TR Pro+™ should not stain clothing once absorbed, however if residual marking does occur, the clothes can be washed as normal to remove the marks given that TR Pro+™ is water soluble

If irritation occurs, discontinue use and consult your physician

Ingredients:

Water, ethanol, triethanolamine, carbomer, beta-glucan (Glucoprime® 0.1%), titanium dioxide

Tissue Repair Ltd is an Australian company, and TR Pro+™ is Made in Australia.

www.trproplus.com



Use the QR code or go to trproplus.com to order TR Pro+™

